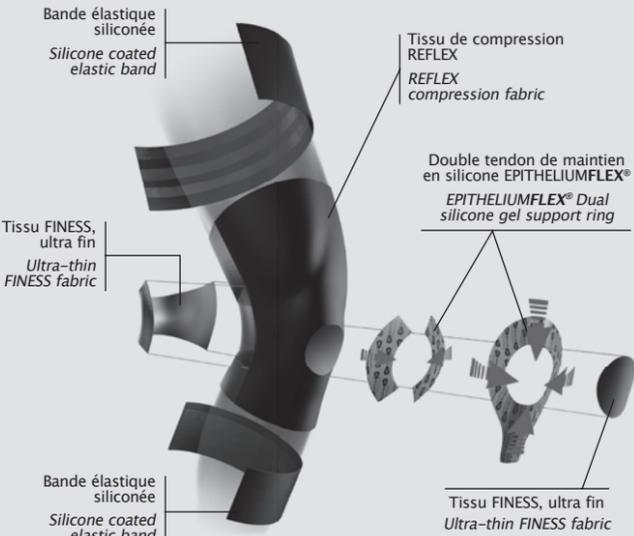
## PATENTED TECHNOLOGY

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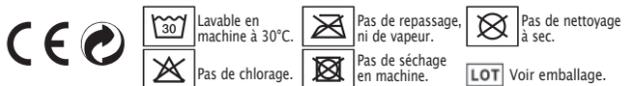
## PHYSIOstrap® Ski

EPITHELIUMFLEX® 03

**INDICATIONS:** Wearing the PHYSIOstrap® Ski helps stabilise the knee joint during board sports which involve repeated flexion and extension movements. It relieves kneecap disorders (patellofemoral pain syndrome, patellar instability, patellar tendinitis etc.), or pain resulting from the residual effects of previous injuries (sprain, ligament or meniscus lesion etc.)

**CONTRAINDICATIONS:** To ensure that blood circulation is not impeded, do not use if you have arteritis, severe varicose conditions, sequelae of phlebitis or oedema (lymphatic/venous) of the lower leg and/or the thigh. Do not use with compression stockings. Do not use on a skin lesion.

### PRECAUTIONS FOR USE:

- To avoid any irritation caused by the silicone elastic bands, the PHYSIOstrap® Ski should not be worn in a static position, particularly with the wearer lying down (sleep) or sitting (e.g. during bus or car journeys, lunch breaks etc.). This PHYSIOstrap® Ski should be put on right before starting board sports activities and worn only during exercise.
- The materials used have passed all the regulatory tests required by ISO standard 10993-1 relating to products that come into contact with the skin. However, if an incident occurs, stop using the product immediately and inform MILLET Innovation.
- If you experience pain, oedema, discomfort or any other abnormal reaction, stop using the product and consult a doctor.

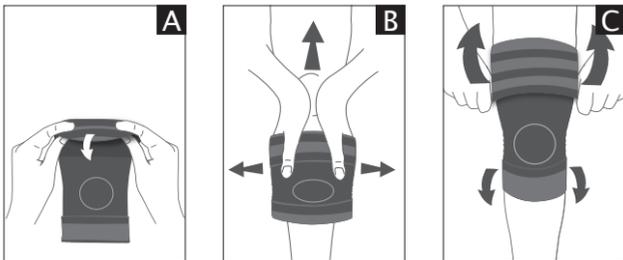
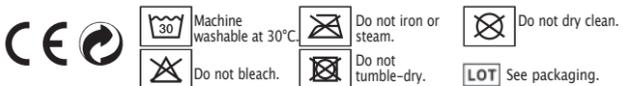
**HOLD:** Do not apply moisturising products to the skin before using the PHYSIOstrap® Ski to ensure optimum hold. **IMPORTANT!** Place the PHYSIOstrap® Ski on the thigh without stretching it. To guarantee good hold, the product must be washed after each use. At the very least, wash the silicone elastic bands with soap and water (without scrubbing them) to remove any dirt.

**WASHING INSTRUCTIONS:** Machine wash at 30°C or hand wash (without scrubbing), leave it to dry naturally. Do not iron.

**ERGONOMIC DESIGN:** The PHYSIOstrap® Ski is a concentrated technology weighing less than 70g, which has been specifically designed for board sports (skiing, snowboarding etc.) Its patented silicone patellar support ring, combined with REFLEX ultra-effective compression fabric, provides optimal kneecap support and knee stability. PHYSIOstrap® Ski is designed to fit either the right or the left knee. It does not slip and does not cause any discomfort at the back of the knee (popliteal fossa). Extra-thin, it can be worn under all types of clothing, even technical clothing (non-compressive).

**COMPOSITION:** 70% polyamide/elastane fabric, 30% silicone.

MILLET Innovation cannot be held liable in cases where the manufacturer's recommendations have not been followed during use. French is the reference language. CE initial marking: 2016.

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### GB HOW TO FIT THE PHYSIOstrap® Ski

- Fold the elastic bands outwards.
- Insert both hands in the PHYSIOstrap® Ski and center the "tendon" on the kneecap.
- Unfold the elastic bands **without tightening the PHYSIOstrap® Ski on your leg.**
- Adjust your PHYSIOstrap® Ski in the standing position **and ensure that the patella support is in a flat position.**

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