PORTUGUÊS

## SEPARADORES com EPITHELIUM™

### INDICAÇÕES

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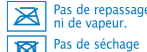
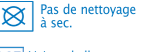
## SEPARATORS with EPITHELIUM™

### INDICATIONS

EPITHELIUM™ separators are used to separate two toes between which there is excessive pressure and friction.

### CONTRAINDICATIONS

- Do not use on a skin lesion.
- Persons with arteritis, diabetes, polyneuritis, peripheral neuropathy, or foot ulcers: consult your physician or chiroprapist before application and monitor your foot during use.

### PRECAUTIONS FOR USE

- Wash before first use.
- Place 1 separator between two toes. The widest part should be located under the foot, with the flat part against the corn.
- Note: To widely separate two toes:
  - Position two separators against one another.
  - Place the assembly obtained between the toes.
- Once the metatarsal corn has been removed by the podiatrist, a separator is put in place. This system prevents the keratome from automatically reforming.

### WHAT IT IS USED FOR

- The separator spreads the toes apart by a few millimetres. It is made of EPITHELIUM™ which has remarkable properties that distribute pressure and prevent friction.
- The flexible material and the shape of the separator, which match the foot, provide immediate comfort.

### CLEANING

Protections may be washed with soap and water. Do not rub, allow to air-dry.

### COMPOSITION: 100% silicone.

The materials used have passed all the regulatory tests required by ISO standard 10993-1 relating to products that come into contact with the skin. If an incident occurs, stop using the product immediately and inform MILLET Innovation.

Millet Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2004.

- |                |                       |                   |
|----------------|-----------------------|-------------------|
| Hand washable. | Do not iron or steam. | Do not dry clean. |
| Do not bleach. | Do not tumble-dry.    | See packaging.    |

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