PORTUGUÊS

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FLEXIBLE PROPRIOCEPTIVE SPLINT (ORTHOSIS) with EPITHELIUMFLEX®

INDICATIONS

Helps to relieve articular arthrosis pain at the root of the thumb by restricting microinstability.

CONTRAINDICATIONS

- Do not use on a skin lesion.
- In the event of severe oedema: look closely at the splint during use. Remove the splint if it is too tight and impedes circulation. When in doubt seek advice from your GP or your pharmacist.

PRECAUTIONS FOR USE

- Wash before first use.
- This product is a day use splint.
- Two versions are available: right hand and left hand splints (stitching is inside the hand splint).
- Do not use near a flame or incandescent body.

WHAT IT IS USED FOR

The EPITACT® flexible proprioceptive splint (orthosis) enables maintaining full function of the hand throughout the day. It is used as a complement to rigid splints which are more suitable for nocturnal use.

WEARING THE SPLINT

Attach the splint to your hand by sliding your thumb into the thumb section. Stitching must be inside the splint. For optimal comfort, adjust the position of the splint by slightly moving the stitching one way or another in the centre of the splint.

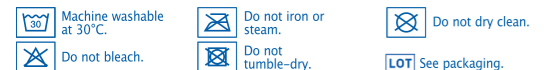
CLEANING

- Machine wash at 30°C.
- Do not use stain removers (acetone, solvents, etc.)
- Leave to dry naturally. Do not iron.

COMPOSITION: 86% polyamide/elastane fabric, 14% silicone.

The materials used have passed all the regulatory tests required by ISO standard 10993-1 relating to products that come into contact with the skin. If an incident occurs, stop using the product immediately and inform MILLET Innovation. Millet Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2013.

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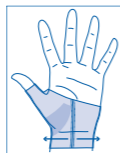
SAMENSTELLING: 86% polyamide/elasthaan, 14% silicone.

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