DANSK

SVENSKA

SUOMALAINEN

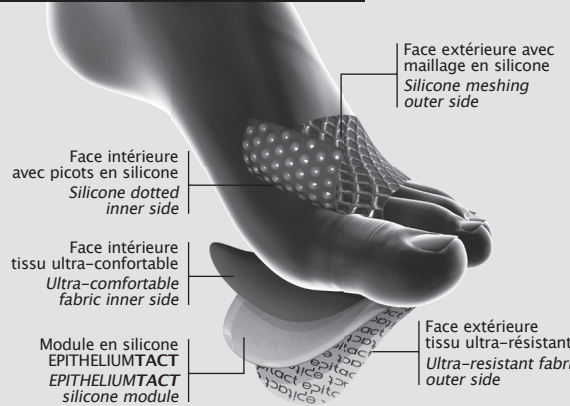
NORSK

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PREVENTIVE PROTECTORS

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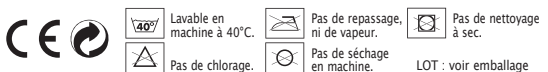
EPITHELIUMTACT 05

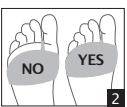
FUNCTIONS:

The EPITHELIUMTACT 05 plantar protectors include a silicone gel just 1.7mm thick which has been specially developed to effectively distribute pressure and absorb friction without creating instability under the forefoot. They therefore relieve pain and prevent the risk of overheating, burning, blistering and callouses.

RECOMMENDATIONS FOR USERS:

These protectors should be used directly on the skin.

- Put on the plantar protector and place the support strap between the big toe and the second toe (1).
- The interlocking silicone dots (on the inner side of the strap) should be in contact with the skin, with the silicone meshing (outer side of the strap) in contact with the sock.
- Adjust the position of the plantar protector at the base of the toes (2) taking care not to pull the strap too tight.
- For optimum support, do not wear the protectors barefoot. Always wear socks over the top. Do not apply moisturising products before use.

WASHING INSTRUCTIONS:

- Machine wash at 40°C or hand wash (without rubbing), leave to dry naturally. Do not iron.

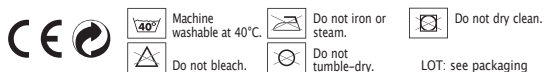
IMPORTANT! To guarantee good hold, the protectors must be washed after each use.

CONTRAINDICATIONS:

- The product has undergone all the regulatory tests required by ISO 10993-1 relative to products that come into contact with the skin. However, if any skin reactions are observed, stop using the product immediately and contact MILLET Innovation.
- Do not use on an open wound.
- Persons suffering from arthritis, diabetes, polyneuritis, and neuropathy of the lower limbs should not use this product.

COMPOSITION: 74% silicone, 26% polyamide/elastane fabric.

MILLET Innovation cannot be held liable in cases where the manufacturer's recommendations have not been followed during use.

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