PORTUGUÊS

ERGOstrap™ SPORT EPITHELIUMFLEX®

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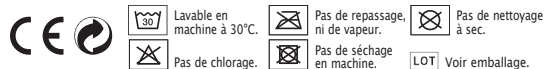
ERGOstrap™ SPORT EPITHELIUMFLEX®

INDICATIONS:

Mild to moderate sprains, chronic ankle instability, resuming sporting activity.

CONTRA-INDICATIONS:

Do not use if you suffer from arteritis, diabetes, polyneuritis, neuropathy of the lower limbs.

USAGE PRECAUTIONS:

Do not place ERGOstrap™ SPORT in direct contact with the skin. ERGOstrap™ SPORT must be worn over socks, high enough to avoid causing discomfort during wear. Do not use when standing (only use while carrying out your activity). If you use compression sleeves on your calves, place them below ERGOstrap™ SPORT. Do not use with compression stockings.

DETAILS:

ERGOstrap™ SPORT supports and stabilises the ankle without restricting movement, thanks to the combined action of heat-mouldable supports around the malleoli and EPITHELIUMFLEX® patented strapping.

ERGONOMICS:

A technological marvel weighing less than 120 g, ERGOstrap™ SPORT has been specially designed for sport. Perfectly adapted to your body shape and extra-slim (no added thickness under the heel), it is very comfortable and can be worn under any type of sock. You will completely forget about it during your sporting activity.

DIRECTIONS FOR USE:

- Heat-moulding: please follow the attached heat-moulding instructions or watch the heat-moulding video which can be found on www.ergostrapsport.com. ERGOstrap™ Sport can be heat-moulded a several times if necessary.
- Positioning: watch the positioning video which can be found on www.ergostrapsport.com.

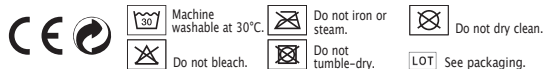
CARE:

Machine-wash at 30°C or hand-wash without rubbing. Allow to dry naturally. Do not iron. Do not expose to heat (> 50°C).

COMPOSITION: 50% polyamide/elastane fabric, 32% heat-mouldable plastic, 18% silicone.

The materials used have passed all the regulatory tests required by ISO standard 10993-1 relating to products that come into contact with the skin. However, if an incident occurs, stop using the product immediately and inform MILLET Innovation.

MILLET Innovation cannot be held liable in cases where the manufacturer's recommendations have not been followed during use. French is the reference language. CE initial marking: 2017.

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